QUICK START – OPERATING YOUR OXYGEN CONCENTRATOR

DANGER-NO SMOKING

 Read the Important Safeguards and observe all Warnings and Cautions on the product and in the Instruction Guide. Follow all operating instructions. Refer to Operation section for more details.

1A OR 1B. Attach the appropriate oxygen accessories to the oxygen outlet. If cannula isn't attached to oxygen tubing, follow manufacturer's instructions to attach.

2. With power switch OFF, firmly attach cord to unit and plug into wall outlet.


1A. Oxygen Tubing Connection: Thread oxygen outlet connector onto oxygen outlet and attach tubing to the connector.

1B. Oxygen Tubing Connection With Humidification: Fill humidifier bottle, thread humidifier bottle wing nut onto oxygen outlet and tighten securely. Attach tubing to humidifier bottle outlet fitting.

Ensure power cord is fully inserted into concentrator IEC connector.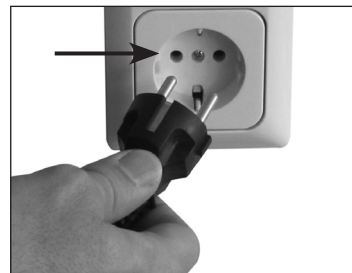

Plug concentrator cord into wall outlet.

3. Press Power Switch to turn unit ON.

NOTE– When first turned ON, all LEDs illuminate briefly and alarm briefly sounds to test system.

4. Ensure flow meter is set to prescribed flow rate.


NOTE–Flow meter ball should be centered on line next to prescribed flow rate.

5. Attach cannula to nose and face. Breathe normally through cannula.


NOTE– Unit operates in “start-up” mode with Startup Low O2 LED illuminated for up to 15 minutes until proper output flow is achieved when Green Good O2 LED illuminates.

6. When finished using the device, press Power Switch to turn unit OFF.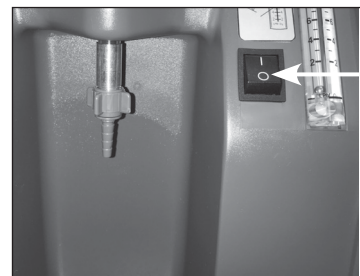

DANGER– Do not leave cannula on bed coverings or chair cushions if unit is turned ON but not in use, the oxygen will make the materials flammable. Turn concentrator OFF when not in use to prevent oxygen enrichment.

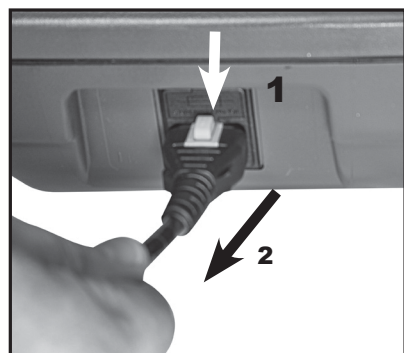7. To remove the line cord:
a. Press the yellow release tab
b. Pull the plug while holding the release tab

Video instruction on unit set up

Video instruction on unit operation

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